# CLEAR-ALIGNER®

## Newsletter 2

### Casos clínicos

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### *Case Reports*

*Cases combining the upper Clear Aligner and lower fixed orthodontics.*

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*All cases were carried out by Dr. Pablo Echarri. All the laboratory work was carried out by Ladent laboratory, and the Scheu Dental Technology materials were used.*

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*This book or any part thereof may not be reproduced, stored in retrieval system or transmitted in any form or by any means electronic, mechanical, photocopying or otherwise, without prior written permission of the publishers.*

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## CASE 1

Ref. 01351B – Anterior deep bite and lower crowding. Treatment with upper Clear Aligner with 3-to-3 alignment and intrusion. 7 steps of upper Clear Aligner. Lower treatment with fixed appliances and one incisor extraction. 10 months of fixed lower treatment. The fixed prosthesis replacement will be necessary.

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Treatment progress with lower fixed appliance and build-ups.

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*Final registers. Replacement of the prosthesis is necessary.*

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## CASE 1

Observe the upper alignment and intrusion, correcting the anterior deep bite.

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## CASE 2

Ref. 01389B – Crowding case in both arches. Upper arch treatment with Clear Aligner with protrusion - 6 steps. Lower fixed orthodontic treatment with stripping - 9 months.

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*Treatment progress - observe upper expansion with Clear Aligner.*

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*Treatment progress - lower expansion to carry out stripping.*

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## *New Agreement Between Ladent and Scheu Dental*

*An agreement has been signed between the LADENT S.L., as a Clear Aligner certified laboratory and SCHEU Dental GmbH, with an objective to promote spread and develop the Clear Aligner Technique. In this new stage, Clear Aligner changes its name to C-A Clear Aligner.*

*The world-wide promotion of this product has already been started with the courses given by Dr. Echarri in Frankfurt on November 11th, and Vienna on November 12th, which had a great acceptance and attendance.*

*Beginning from November, Ladent will offer the training and PREMIUM CA certificate to Clear Aligner laboratories all over the world, and is exclusive distributor of Clear Aligner products for Spain.*

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*All rights reserved*