


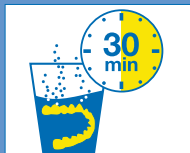

Application

- 1 Dissolve the content of a bag in 150 ml of water. Rinse the appliance/splint with fresh water and put it in the solution.
- 2 Leave the appliance/splint for about 30 min. in the solution. Do not keep mixed solution longer than 3 to 4 days.
- 3 Any remaining sediment can be removed with a toothbrush and fresh water.
- 4 Apply CETRON® care spray to the appliance/splint before inserting it - for a refreshing and lemony flavor!



cetron®



Cleansing and Care

for orthodontic appliances,
splints and dentures

Provided by:

www.scheu-dental.com

SCHEU

GB 01/2.000/02/13 G REF PM0011.02



SCHEU
Dental Technology



cetron®

Best cleansing and care

Due to contact with food, drinks, pharmaceuticals and individual mouth environment, splints, orthodontic appliances and dentures may change their colour, taste and smell. Sediments can stick to the surface.

With CETRON® you purchase a good cleansing agent with a fast cleansing and long lasting care effect.

All dentures and orthodontic devices can be cleaned, freed from sediments and refreshed.



The care spray for application directly before inserting a splint or an orthodontic device.



Practical 15g portion bags for patient use.
The bag's content is dissolved in water in the cleansing box.