CLEAR-ALIGNER®

Newsletter 12

Casos clínicos

Case Reports

CLEAR-ALIGNER®

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All cases were carried out by Dr. Pablo Echarri. All the laboratory work was carried out by Ladent laboratory, and the Scheu Dental Technology materials were used.

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© 2013 Centro de Ortodoncia y ATM, Ladent, SL
All rights reserved

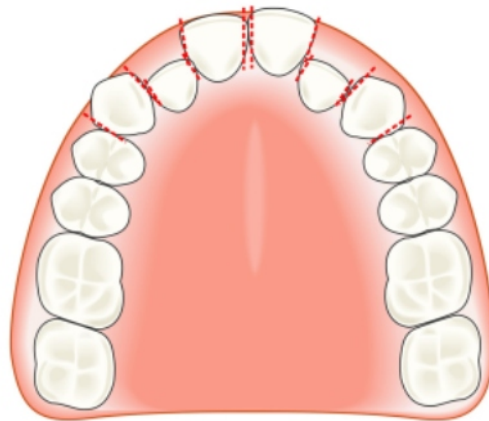
This book or any part thereof may not be reproduced, stored in retrieval system or transmitted in any form or by any means electronic, mechanical, photocopying or otherwise, without prior written permission of the publishers.

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
CA CLEAR ALIGNER AND PROGRESSIVE STRIPPING TECHNIQUE

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
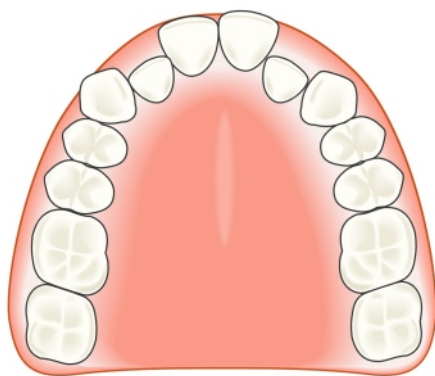
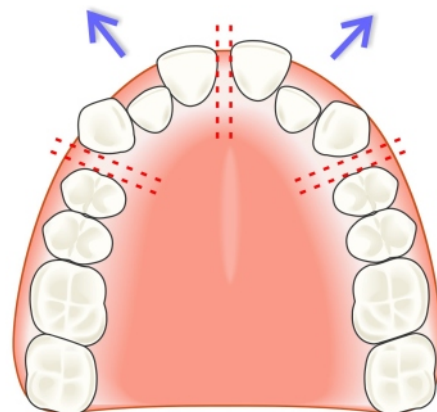
Use of Clear Aligner in oblique-lateral expansion and space opening: between canines and first bicuspids, and between central incisors.

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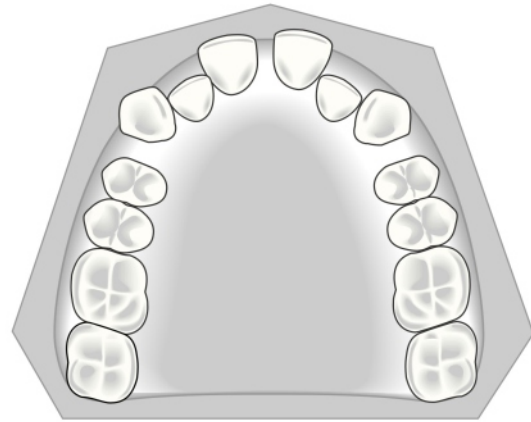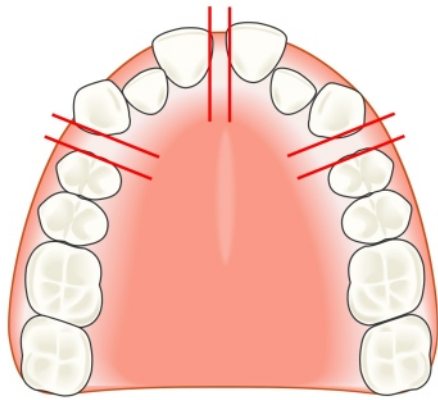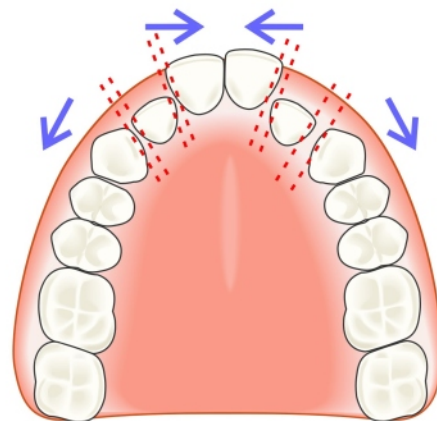
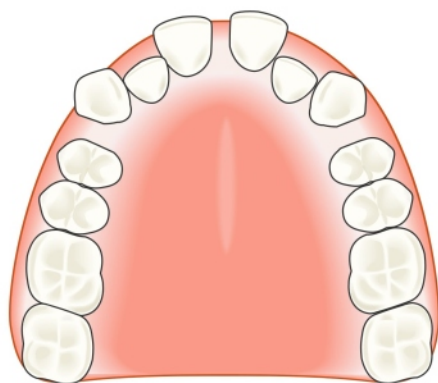
Carry out the stripping on distal surfaces of canines, on mesial surfaces of first bicuspids, and on mesial surfaces of central incisors, and take new impressions, keeping the previous Clear Aligner.

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

Distalize the canines, and mesialize central incisors, until you open the spaces on mesial and distal of lateral incisors.

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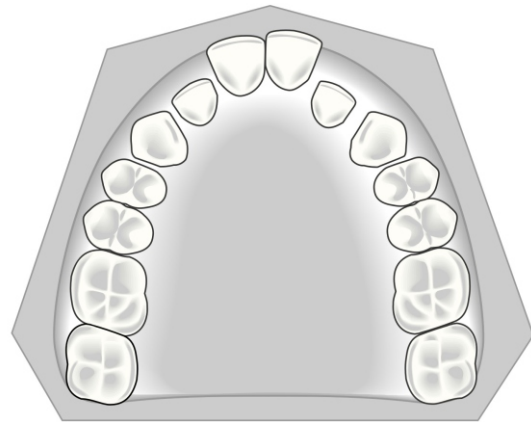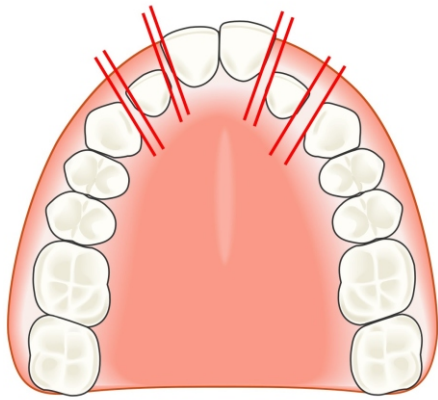
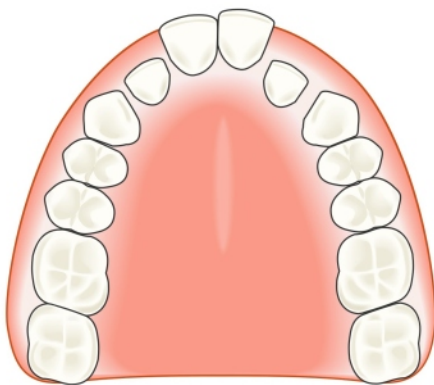
Carry out the stripping of distal surface of central incisors, mesial and distal surfaces of lateral incisors, and mesial surface of canines, and take new impressions, keeping the previous Clear Aligner.

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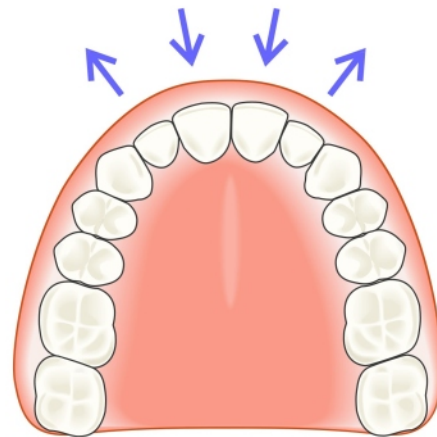CASE 1

Ref. 00535 – Class I with crowding and rotations in both arches. Deep bite. Upper arch – lateral-oblique expansion, intrusion of incisors, stripping and C-A Power Grips in upper right canine. 9 steps. Lower arch – lateral-oblique expansion. 5 steps.

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CASE 2

Ref. 01259 – Molar Class III without 2 lower bicuspids. Lateral-oblique expansion and stripping in both arches. Forced extrusion of the lower left second bicuspid. Upper arch – 10 steps. Lower arch – 10 steps.

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CASE 3

Ref. 01291 – Class I with crowding and rotations in both arches. Treatment with expansion and stripping in both arches. Upper arch – 7 steps. Lower arch – 11 steps.

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CASE 4

Ref. 01297 - Class I with crowding and rotations in both arches. Upper arch - expansion in 6 steps. Lower arch - Expansion and stripping in 10 steps.

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