  
CLEAR ALIGNER

20

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*All cases were carried out by Dr. Pablo Echarri. All the laboratory work was carried out by Ladent laboratory, and the Scheu Dental Technology materials were used.*

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© 2014 Centro de Ortodoncia y ATM, Ladent, SL  
All rights reserved


This book or any part thereof may not be reproduced, stored in retrieval system or transmitted in any form or by any means electronic, mechanical, photocopying or otherwise, without prior written permission of the publishers.

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
## Fabrication procedure of anterior build-ups in upper incisors using CA Power Grips 8 and 9

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
The CA Power Grip 8 BUAS (Build-Up Anterior Small), and CA Power Grip 9 BUAL (Build-Up Anterior Large) are used in palatal surfaces of the upper incisors in cases of anterior deep bite to make the intrusion of upper and lower incisors easier, carried out both with CA Clear Aligner or with fixed orthodontics.

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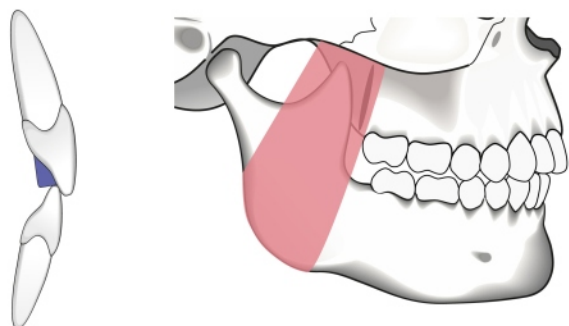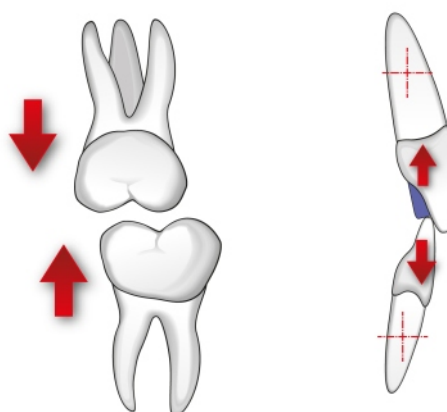
**Figure 1.** Scheme of CA Power Grip 8 and 9.  
**Figure 2.** Action of CA Power Grip 8 and 9.

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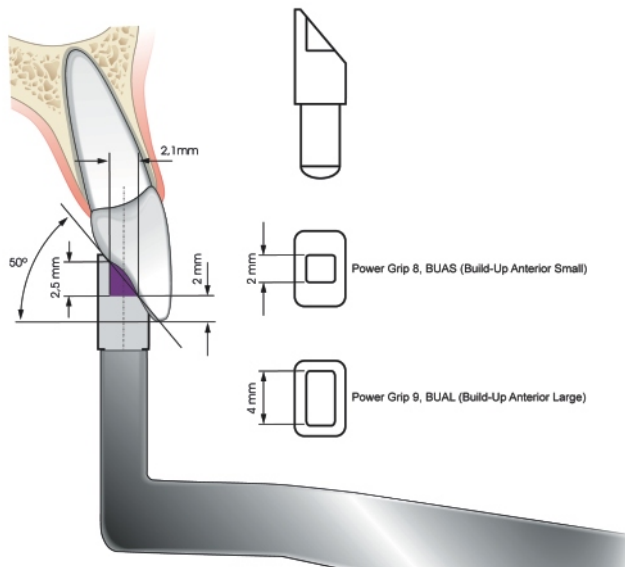**Figure 3.** Positioning of the CA Power Grip 9 2 mm from the incisal border.

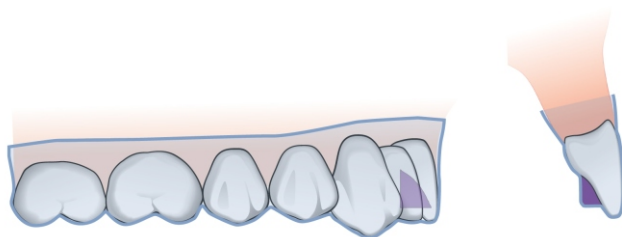**Figura 4.** CA Power Grip 8 y 9 con CA Clear Aligner.

**Figure 4.** CA Power Grip 8 and 9 with CA Clear Aligner.

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**Figure 5.** Isolate the working field with the separator, cotton rolls and the aspirator.

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**Figures 6 & 7.** Clean the palatal surfaces of the upper teeth with the prophylactic paste.

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**Figure 8.** Acid etching for 30 seconds.  
**Figure 9.** Remove the orthophosphoric acid with the aspirator.

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**Figures 10 & 11.** Rinse each tooth for 10 seconds each, and dry.

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**Figure 12.** Acid etching is finished.  
**Figure 13.** Apply Primer.  
**Figure 14.** Cure the Primer with the light.

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
**Figures 15 & 16.** CA Power Grip on the CA Power Grip support.

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**Figures 17-19.** Fill the CA Power Grip with the composite.

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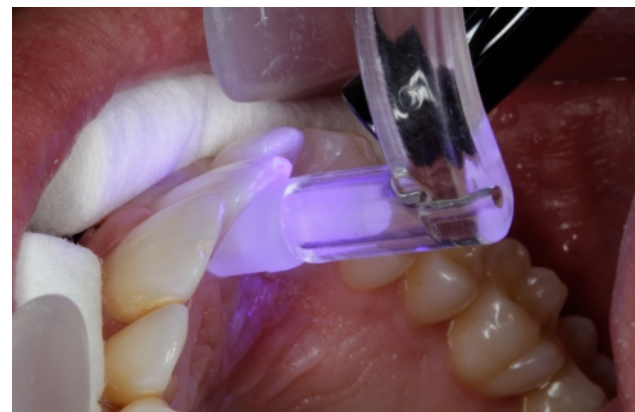**Figure 20.** Stick the CA Power Grip to the palatal surface of the upper central incisor.

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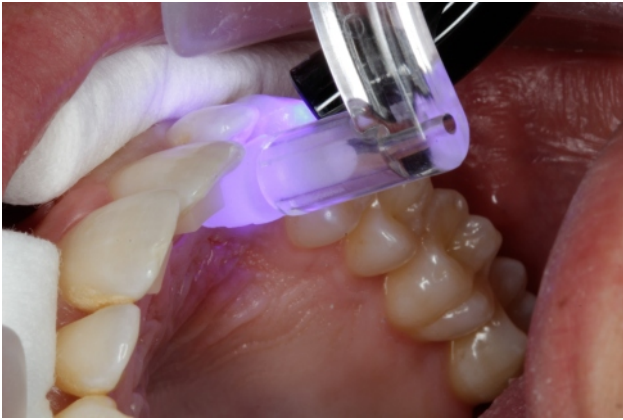
**Figure 21.** Cure the CA Power Grip of upper central incisor. Hold firmly the CA Power Grip on the tooth without moving it, and light cure it during 30 seconds.

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**Figure 22.** CA Power Grip at the upper central incisor.

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**Figures 23 & 24.** Fabrication of the CA Power Grip in the lateral incisor and its light curing.

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**Figure 25.** Anterior build-ups in all four incisors.

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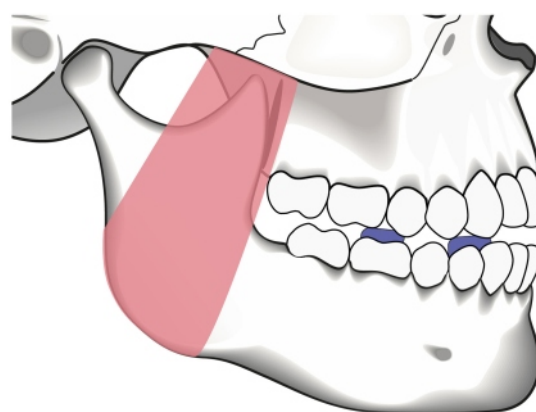
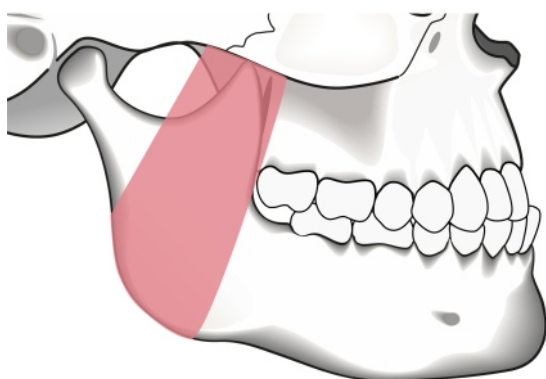
## Fabrication procedure of posterior build-ups in lower molars and bicuspids using CA Power Grips 6 and 7

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The CA Power Grip 6 BUB (Build-Up Bicuspid) is used in fabrication of build-ups in bicuspids. It has a cylindrical shape with a circular base of 3 mm diameter, and it is 2 mm high. The CA Power Grip 7 BUM (Build-Up Molar) has a cylindrical shape, with ovoid base whose diameters are 3 mm and 5 mm, and it is 2 mm high. They are indicated to increase the vertical dimension when there are interferences in correction of the anterior teeth in cross bite. They can be used in treatments with both CA Clear Aligner and fixed orthodontic appliances.

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**Figure 26.** The CA Power Grip 6 & 7 are indicated in anterior cross bite.

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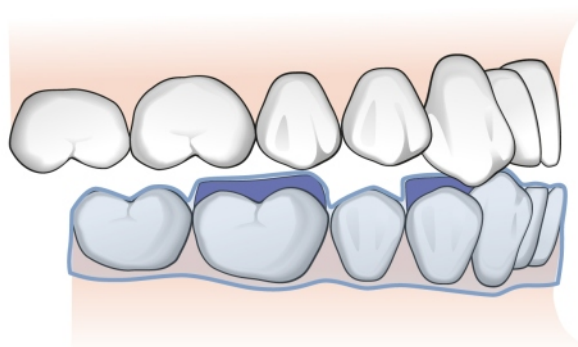
**Figure 28.** Insertion of CA Power Grip 6 & 7.

**Figuras 29.** CA Power Grip 6 y 7 con CA Clear Aligner.

**Figure 29.** CA Power Grip 6 & 7 with CA Clear Aligner.

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**Figure 30-34.** A patient with Class I malocclusion with crowding in both jaws and cross bite in upper lateral incisors.

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**Figure 35.** Upper brackets are bonded. The patient shows occlusal interferences with brackets of upper lateral incisors. Posterior build-ups are indicated to increase the vertical dimension and to avoid occlusal interferences, facilitating the alignment of upper teeth.

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**Figure 36.** Carry out prophylaxis in all teeth. Isolate the working field with separators and cotton rolls. Carry out orthophosphoric acid etching of the lower first molar and first bicuspid during 30 seconds.

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**Figure 37.** Rinse each tooth during 10 seconds, dry and apply Primer on the lower right first molar and first bicuspid.

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**Figure 38.** Apply CA Power Grip 6 BUB to the lower right first bicuspid. Hold the CA Power Grip on the tooth without moving, and cure it with light during 30 seconds.

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**Figure 39.** CA Power Grip 6 BUB finished in the lower right first bicuspid and CA Power Grip BUM finished in the lower right first molar.

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**Figure 40.** Isolate working field on the left side.

**Figure 41.** Acid etching of the lower left first molar and the first bicuspid during 30 seconds.

**Figure 42.** Rinse, dry and apply primer to the lower left first molar and first bicuspid.

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**Figure 43.** Build-up of the lower left first molar is finished.

**Figure 44.** Build-up of the lower left first bicuspid is finished.

**Figure 45.** Build-ups of bicuspids and molars are finished.

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**Figure 46-48.** Increase of the vertical dimension using build-ups in molars and bicuspids.

CLINICAL PAPER

CLEAR ALIGNER