CLEAR ALIGNER

27

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CLASS II TREATMENT
WITH DISTALIZATION USING
CA[®] DISTALIZER

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All cases were carried out by Dr. Pablo Echarri. All the laboratory work was carried out by Ladent laboratory, and the Scheu Dental Technology materials were used.

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© 2016 Centro de Ortodoncia y ATM, Ladent, SL
All rights reserved

This book or any part thereof may not be reproduced, stored in retrieval system or transmitted in any form or by any means electronic, mechanical, photocopying or otherwise, without prior written permission of the publishers.

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CLASS II TREATMENT WITH DISTALIZATION USING CA[®] DISTALIZER

PROTOCOL FOR CA[®] DISTALIZER USE IN MAXILLA IN CLASS II CASES

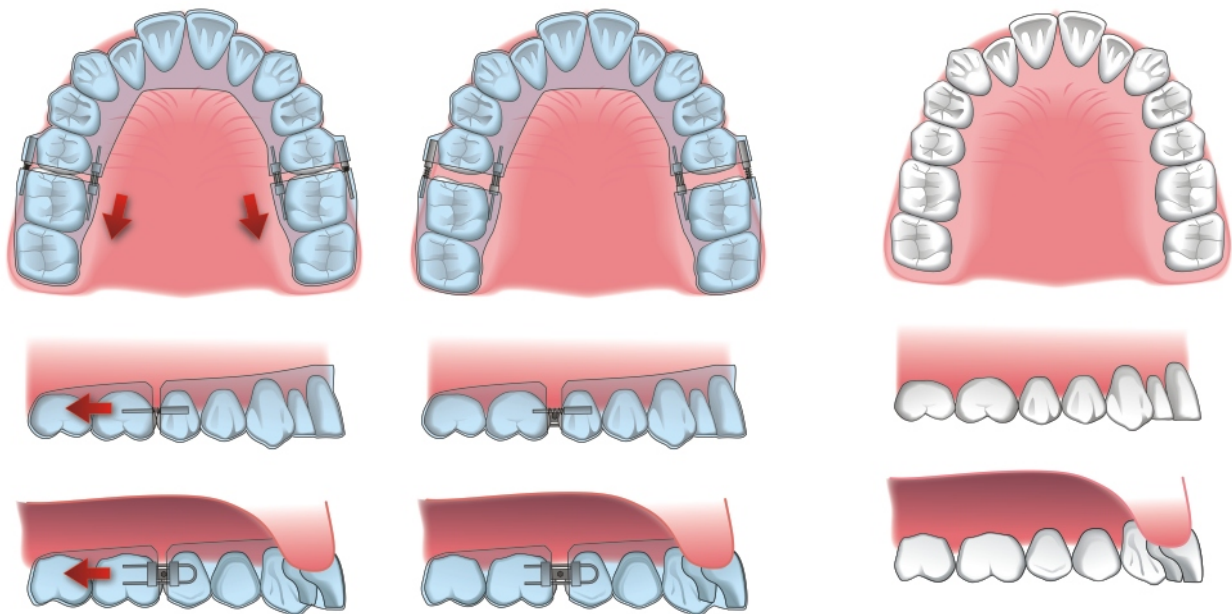
Figure 1.

Ca[®] Distalizer Kit with VECTOR[®] 350 Screw is used to carry out the CA[®] Distalizer. The CA[®] Distalizer Kit is designed by Dr. Pablo Echarri, and it consists of unidirectional VECTOR[®] 350 Screw which is positioned on palatal side, and a tube, a sectional arch wire and an open-coil spring, which are positioned on the labial side.

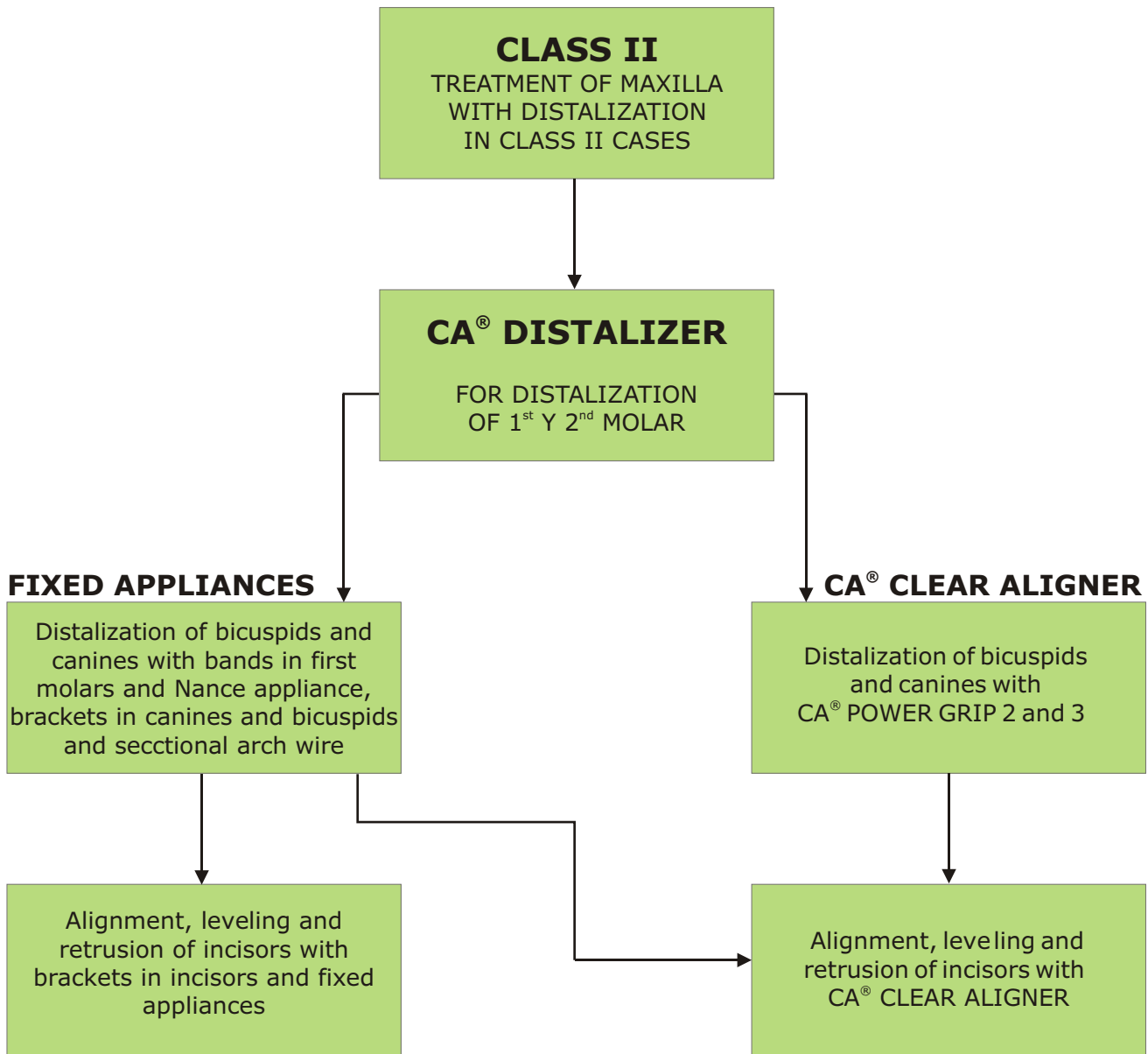
Figure 2.
Protocol for the use of CA[®] Distalizer in Class II.

CA[®] Distalizer can be used to distalize the first and second molar and to finish the whole treatment with CA[®] CLEAR ALIGNER or to combine it with fixed appliances, depending on the preferences of the clinician. The CA[®] CLEAR ALIGNER is used for approximately 2 months for molar distalization, and then the treatment is continued with fixed appliances, or the fixed appliances can be used only for distalization of bicuspid and canines, and then the brackets should be removed and the treatment finished with CA[®] CLEAR ALIGNER.

INDICATIONS

The maxilla distalization with CA[®] Distalizer is indicated in:

- Molar Class II cases, until the molar Class I is achieved.
- Brachyfacial and dolichofacial patient, because the mandible rotates 1° in clock-wise direction for each 3 mm of molar distalization.
- Anterior deep bite, because mandibular clock-wise rotation reduces the overbite with distalization.
- It is better if the third molar is absent or with indicated extraction. The distalization carried out immediately after the extraction of the third molar is quicker due to a high density of osteoblasts and osteoclasts in the zone.

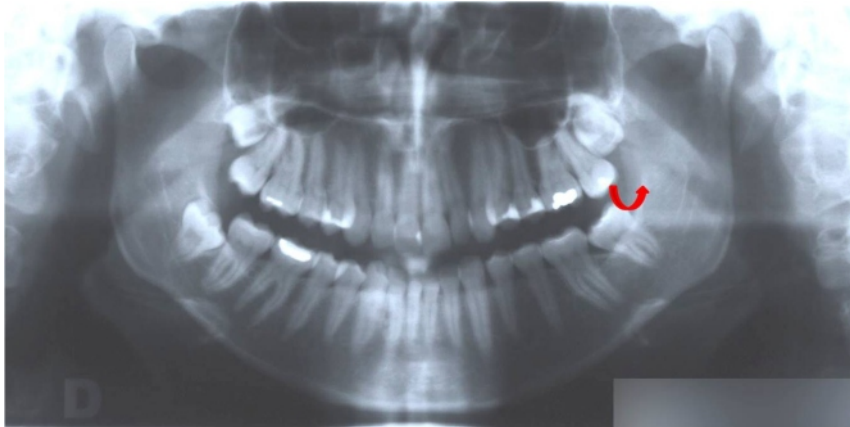
Figure 3.

If the third molar is present, the distalization provokes distoclination of the molar and there is a risk of root resorption of the second molar.

LIMITATIONS

- Dolichofacial patient.
- Anterior open bite.
- Presence of the third molars.

FAVORABLE CONDITIONS

- The molars should **NOT** present more than 15° of mesial rotation.
- The molars should **NOT** present more than 15° of mesial inclination.
- The molars should present normal overjet.
- There must be some available space in the alveolar bone before the distalization.

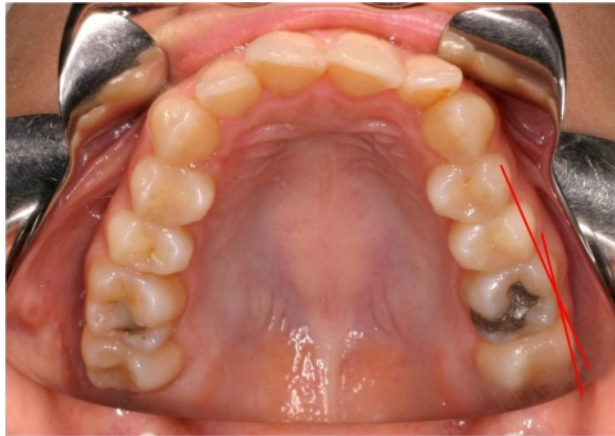
Figure 4.
The molar rotation is the angle between the tangent of the labial surface of the molar and the dental arch line.

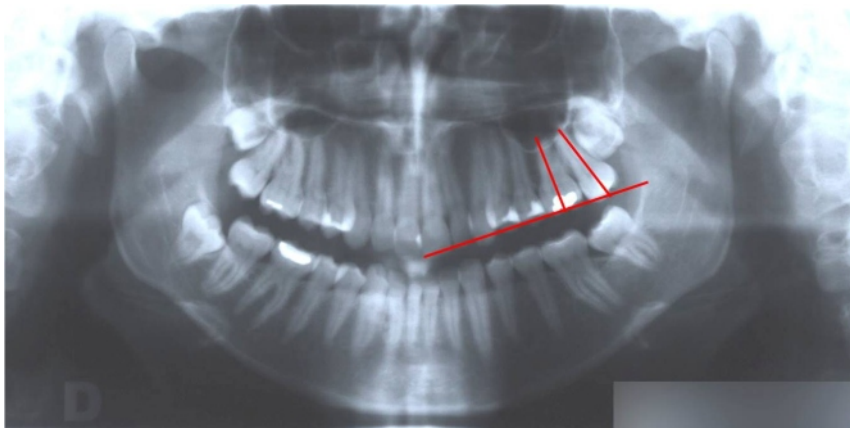
Figure 5.
Inclination of the molar is the angle between the axis of the first or second molar and the occlusal plane.

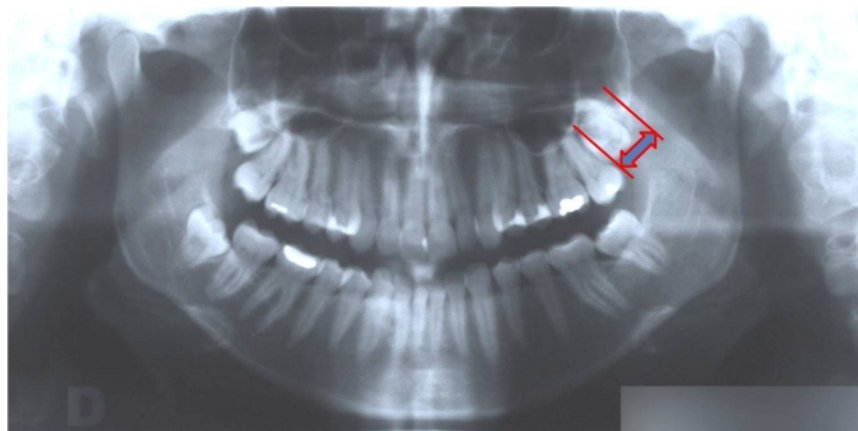
Figure 6.
To evaluate the extraction space, the distance between the distal side of the second molar and tuberosity tangent is evaluated.

TIPS

If molars are in mesial rotation, it might be convenient to bond CA® POWER GRIP n°1 on the labial surface of the molar before taking the impression.

Before adapting the CA® Distalizer, the gingival part of the CA® POWER GRIP should be blocked out, since the used material is not elastic and it would not be possible its insertion. Also, the palatal and distal sides of the zone of the molars should be also blocked out to allow the distal rotation during the distalization movement.

ADVANTAGES

The CA® Distalizer advantages are:

- It has better vertical control of the molars during the distalization.
- It avoids occlusal forces which limit the distalization.

ACTIVATION

The rhythm of activation should be 1 turn each 2-3 days, until the molar Class I is achieved.

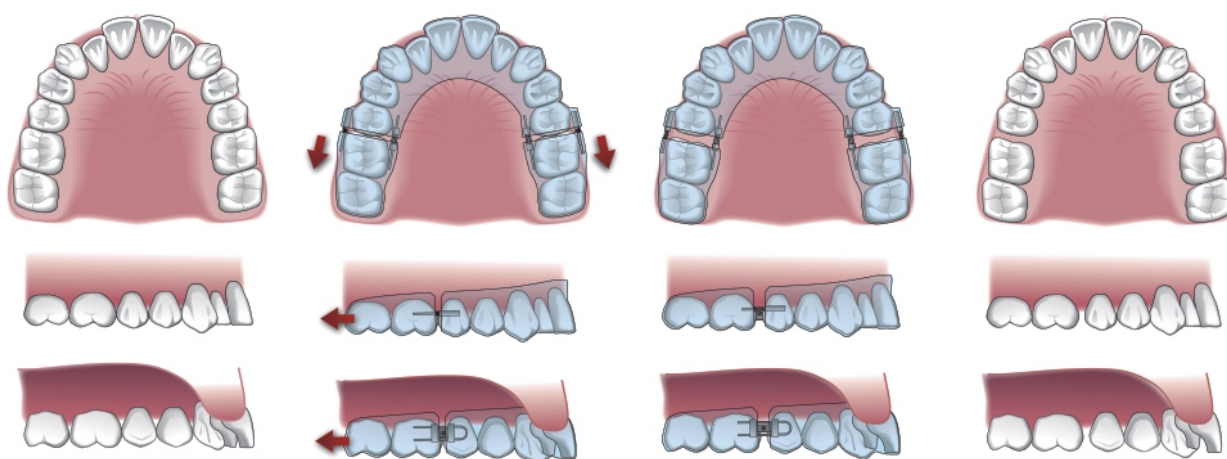
Figure 7.
CA® Distalizer Kit with VECTOR® 350 Screw inserted in CA® CLEAR ALIGNER.

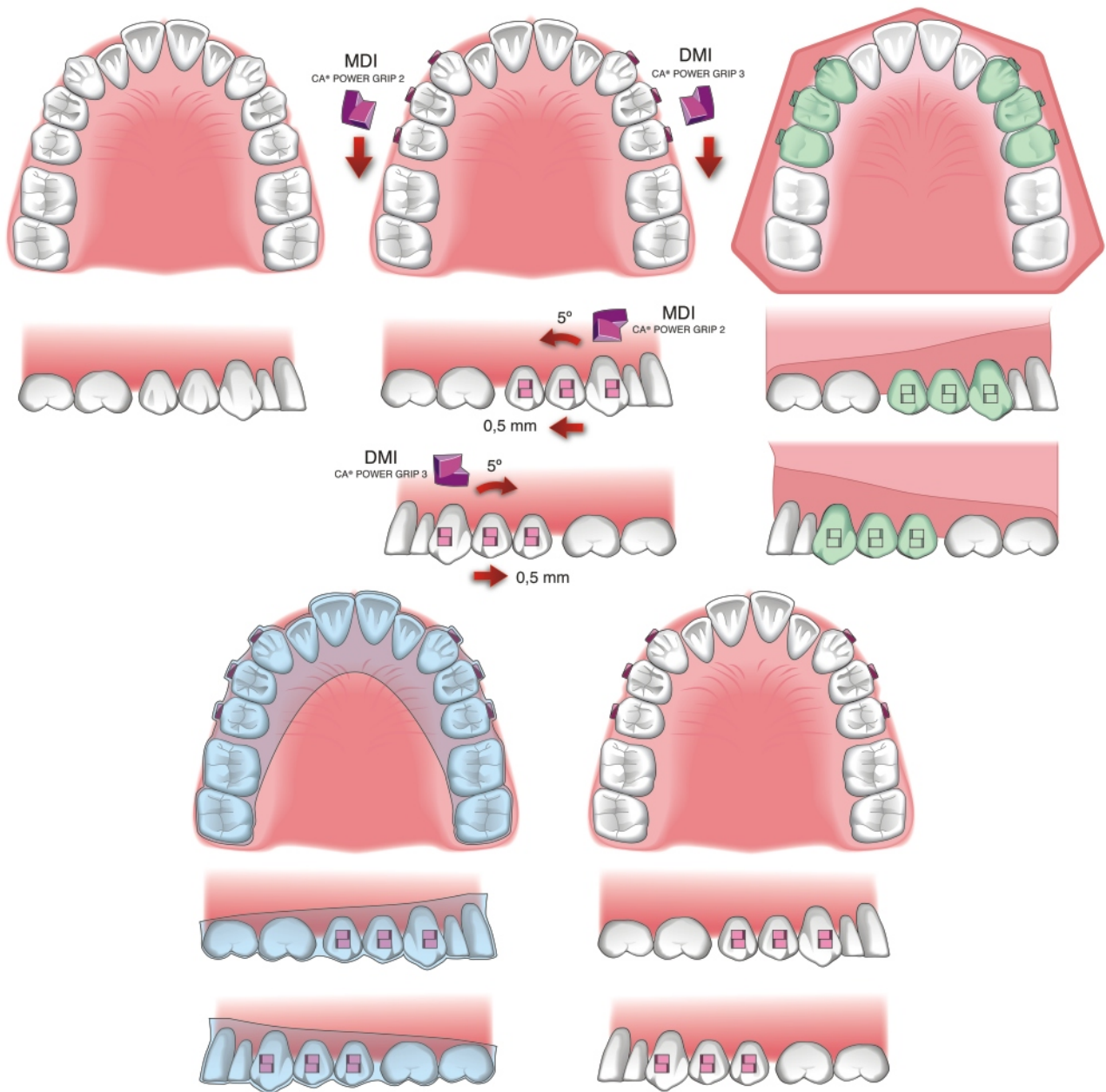
Figure 8.

Canine and bicuspids distalization is carried out simultaneously if it is not necessary to correct rotation, to align or to level these teeth. If it is necessary, they should be distalized one by one, and the malpositioned ones should be corrected

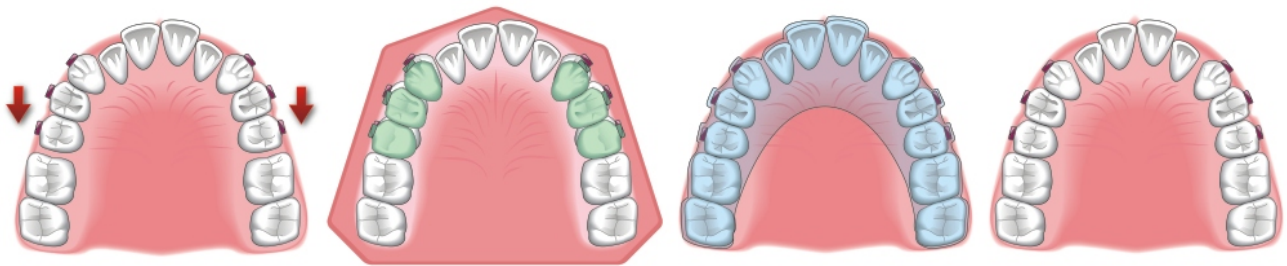
Figure 9.
Continuation of canine and bicuspid distalization.

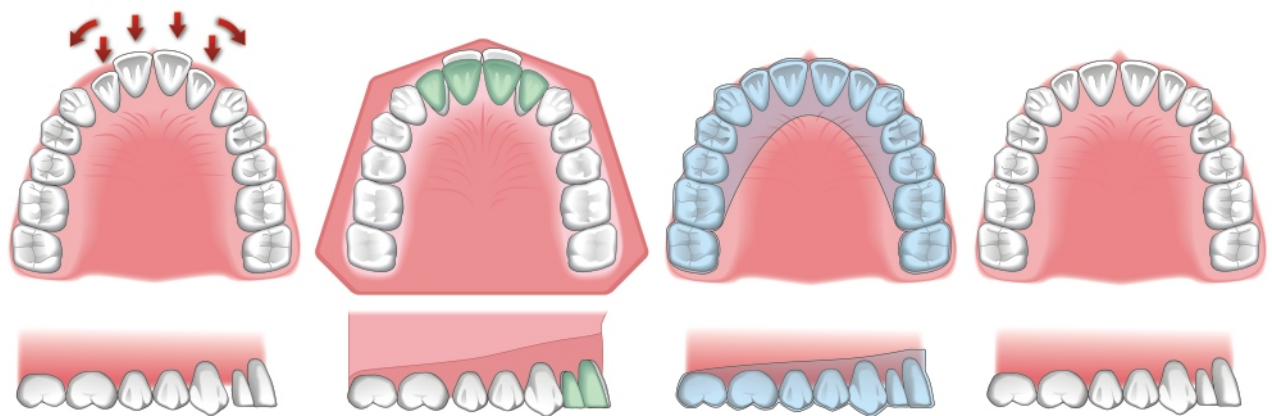
Figura 10.
When the canine Class I is achieved, the incisors alignment is carried out with progressive set-ups. In continuation, the final detailing is carried out.

CLINICAL PAPER

CLEAR ALIGNER